

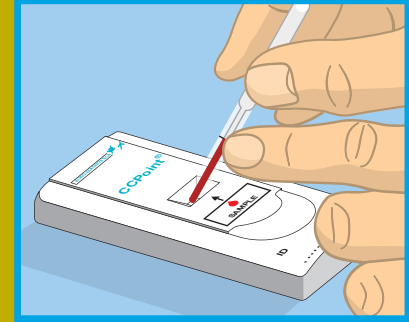
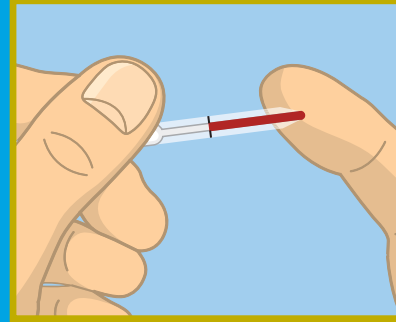
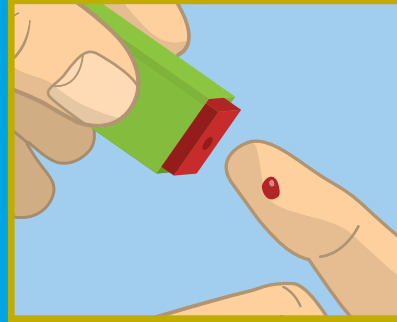
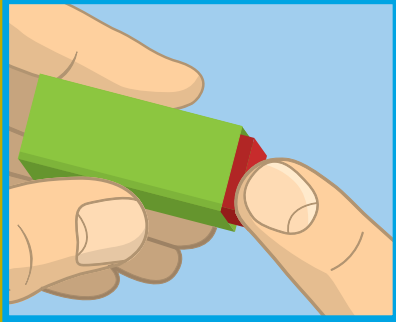
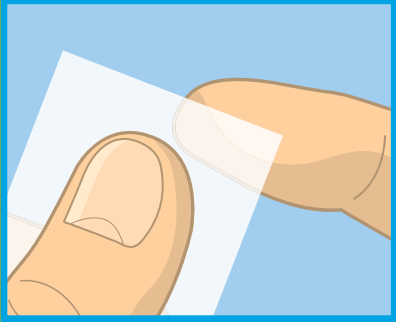
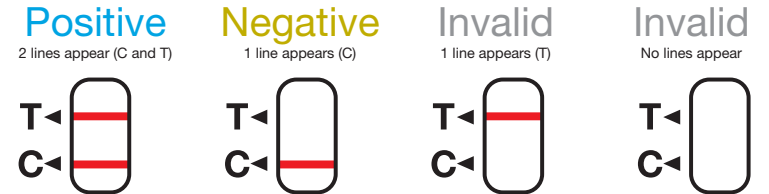
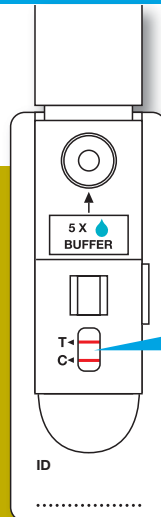


1. Please carefully read the instructions before use.
Select a finger and clean with the swab provided.
2. Place the safety lancet on the finger. Push the top of the lancet to pierce skin with a small pin prick.
3. Pull the lancet away. Gently massage the finger in the direction of the fingertip to produce a droplet of blood.
4. Hold the pipette **HORIZONTALLY** and touch the tip to the blood coming from the pin prick. The pipette will automatically fill with blood to the marked black line.
5. Place a finger over the vent hole (black line) and squeeze the bulb to transfer the blood sample into the CCPoint® **SAMPLE PORT**.

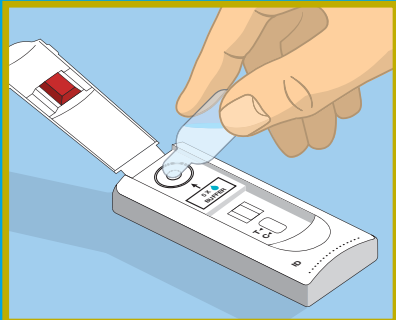

6. **WAIT 30 SECONDS** and then lift the CCPoint® lid.
7. Add **5 DROPS** of **BUFFER SOLUTION (BUF)** into the marked **BUFFER PORT** and allow to stand for **10 MINUTES**. **DO NOT CLOSE THE LID.**
8. After leaving the CCPoint® unit to stand for 10 minutes view the indicator window. The four possible end results are shown below.

If a **POSITIVE** result is displayed you may be in the early stage of Rheumatoid Arthritis (RA) and you should seek an appointment with your Doctor.

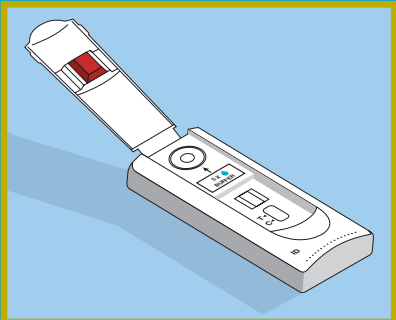

The test is positive even if the T-line is lighter in colour than the C-line.

It has been shown that not all patients with Rheumatoid Arthritis have a positive CCPoint® result. If you get a negative result, but have symptoms, you should seek an appointment with your Doctor. If you get an invalid result we recommend that a new test is performed.