**Strips for semi-quantitative determination of glucose in whole blood**

**General remarks**

The patient can perform the test by himself; test result must however be put under the constant control of a physician or a medical staff to define together the timetable of the determinations, the interpretation of results and the therapy.

**Description**

Visual Glucose permits the semi-quantitative determination of glucose concentration in human capillary blood. Glucose concentration is estimated by comparison with comparative card in the range 40-500 mg/dl. Comparative card is included in the box.

**Intended use**

For rapid determination of blood glucose in suspected diabetics in suspected hypoglycaemic states and in diabetic patients where no quantitative method for self monitoring is available. For a correct diagnosis of diabetes it is necessary to perform the determination of blood glucose with a quantitative system.

**Executing the test**

Required accessories: Sterile gauze, a timer or watch displaying seconds, lancing device and sterile lancets if not included in the box.

If the sterile auto-lancets are included, follow the instruction of use displayed below.

**Preparation:** Prepare a chronometer or a clock to read seconds. Pull out comparative card from pack. Wash hands with hot water and soap and dry them accurately. If disinfectants are used, before performing puncture of fingertip, be sure that this part is well dried. Open the vial and pull out reactive strip immediately before execution of the test.

**Puncture of the fingertip:** read instructions carefully and follow pictures.

1. Be sure that protective cap is not removed. Load lancet device pushing protective cap inside it.
2. Twist off the protective cap as described in the picture and then take out when it easily turns.
3. Press the fingertip on the part of lancet device where protective cap is just removed, be careful to position it not too close the nail and not in the centre of fingertip. Then strongly press the little lever to hear a click. In the case that puncture was not performed, it is possible to re-load the lancet device housing-back the needle by pressing the protective cap into its original site. Lancing device is a single use device: do not use it to perform a second puncture.

**Measurement:** Wipe away the first blood drop with a sterile gauze.

4. Gently press the finger tip, without force and avoiding squeezing and milking, to obtain a second blood drop and approach the finger to the strip. When there is a hanging drop (about 10µ - its diameter must be of 5 mm), put it on the white reactive square, in the middle of the green zone, without actually touching the white reactive square with the finger.
  5. Be sure that blood drop completely covers reactive area of the strip. Wait about 50 seconds for the chemical reaction to occur.
  6. Turn the strip over.
  7. Within next 25 seconds compare the developed colour on the strip with each coloured area of the comparative card by positioning the backside of the strip on the rear of the comparative card in correspondence of the holes. Find the closest matching colours.
- Read on the comparative card the written respective glucose concentration.  
Vial and cap must be discarded properly.

**Interpretation of results**

Test reading: Visual reading is affected by personal evaluation and can change individual by individual.

Since the test result is a semi-quantitative judgement based on colour, shade and intensity, it is useful to get the opinion of a second person on final result. It is important to make the colour comparison within next 25 seconds, otherwise the colour will be going to be darker and the evaluated concentration will be higher than the real one. Each colour block corresponds to a glucose value. Lightest colour is glucose 40 mg/dl. A reactive strip with a colour lighter than 40 mg/dl has to be considered as a glucose level lower than 40 mg/dl.

A reactive strip with a colour darker than 500 mg/dl has to be considered as a glucose level higher than 500 mg/dl. Normal Blood Glucose Values: at fast normal glycaemia values in a non-diabetic person are: 70-110 mg/dl (3,9 – 6,1 mmol/l) on whole blood.

Important! Caution is recommended in the interpretation of glucose values below 50 mg/dl or above 250 mg/dl. Consult a physician as soon as possible if such values are obtained.

**Principle of test**

Each strip has a reactive area on which Glucose-oxidase reacts with D-glucose present in blood and generates hydrogen peroxide which, due to the presence of peroxidase, reacts with the chromogen.

The colour intensity is proportional to the concentration of glucose present in the blood drop.

**Limitation**

- The system is intended for analysis with capillary blood. Do not use serum or plasma.
- If the glucose value obtained does not correspond to the sensations felt by the patient and/or to the knowledge he usually has of his health conditions and symptoms, the test needs to be repeated.
- A physician must be however consulted.
- Capillary blood glucose values may be significantly lower than "real glucose levels" in the hyperglycaemic hyperosmolar state, with or without ketosis.
- Hands and fingers contaminated with sugar from foods or beverages may cause false elevated results. Wash hands in warm water, rinse and dry before testing.
- Severe dehydration and excessive water loss may cause inaccurate low results. Dehydrating factors include vomiting, diarrhoea and the intake of certain drugs such as diuretics.
- Reducing agents e.g. ascorbic acid at normal concentrations, do not interfere. If high quantities are intaken, abnormally low results could be obtained.
- Samples with abnormally high lipid concentration may affect the result.
- Colour-blind individuals should not use visual colour procedures.
- Abnormal haematocrit values (higher than 55% or lower than 35%) can affect the results with visual glucose strips.
- Visual glucose strips are not suitable to perform determinations on neonatal blood samples.

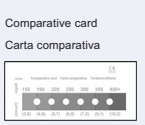
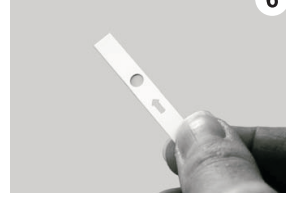
**Warning**

Since the visual glucose is a semi-quantitative test, a physician should be contacted after the result before any dietetic or therapeutic measures are applied.

Strips have to be stored between 10°C and 35°C (50°F e 95°F) and in their original capped vial. Do not expose to direct sunlight. Use within and not after expiry date indicated on the pack and within 3 months from first opening.

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Manufacturer: Biochemical Systems International S.r.l.